


Notice d'utilisation Pince-nez

Le pince-nez FIM MEDICAL est un accessoire développé spécifiquement pour les spiromètres SPIROLYSER® SPL10® USB et SPIROLYSER® Q13®.

Attention : Utilisation uniquement par un médecin ou un professionnel de santé.

Attention : Ne jamais confier le pince-nez aux enfants.

Attention : Le pince-nez est à usage unique.

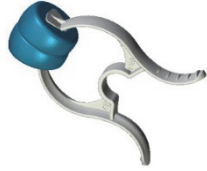

Installation :

- Sortir le pince-nez de son emballage
- Ne pas toucher la partie en mousse avec les doigts mais le tenir par le corps
- Positionner le pince-nez sur votre nez afin de boucher les narines


Utilisation :

- Changer systématiquement le pince-nez à usage unique entre chaque patient

Elimination des déchets :

- Les accessoires souillés doivent faire partie d'une collecte sélective pour déchets biologiquement souillés (DASRI).

Caractéristiques :

Pince-nez avec mousses biocompatibles		Usage unique
Date limite d'utilisation	Voir étiquetage du carton	
Stockage / transport	T° : entre 0°C et 50°C Humidité : 75% max.	
Classe médicale	Classe IIa	

Marquages :



CE
0459

Marquage CE directive 93/42/CEE + N° de l'organisme notifié



Consulter les documents d'accompagnement




LOT



Identification du fabricant



Date limite d'utilisation

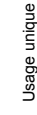

Usage unique

Instructions Nose clip

The FIM MEDICAL nose clip is an accessory developed specifically for SPIROLYSER SPL10® USB and SPIROLYSER® Q13® spirometer.

Caution: U.S.A. Federal Law restricts this device to sale by or on the order of a physician.

Caution: Keep out of reach of children.

Caution: The nose clip is single-use.



Installation

- Remove nose clip from the packaging
- Do not touch the foam pads
- Place the nose clip on the nose so as to block the nostrils


Use

- Systematically change the single-use nose clip between patients

Disposal

- Used or dirty accessories must be disposed of with biological waste (DASRI).

Features

Nose clip with biocompatible foam pad		Single-use
Expiry date	See cardboard box label	
Storage/transport	T° : 0°C - 50°C Humidity : 75% max.	
Medical class	Class IIa	

Labels:



CE
0459

CE Marking Directive 93/42/CEE + N° of notified body



Consult the accompanying documents



LOT



Manufacturer identification



Expiry date



Single use