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
## Instruction sheet

### Pipette and Trapper for TABATABA® V2

FIM MEDICAL pipettes and trappers are accessories developed specifically for the TABATABA® V2 device (analyser of carbon monoxide (CO) in expired air).

**Caution:** U.S.A. Federal Law restricts this device to sale by or on the order of a physician.

**Caution:** Always use a pipette and trapper with the TABATABA® V2.

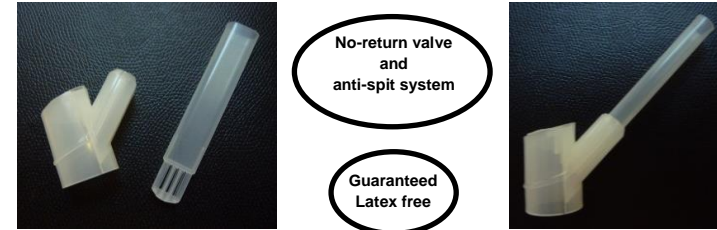#### Installation:

- Remove the trapper and pipette from the packaging
- Do not touch the mouthpiece of the pipette with the fingers
- Insert the pipette into the trapper and connect the set to the device

#### Use:

- Systematically change the single use pipette between patients
- Systematically change the single use trapper between patients
- **Caution:** risk of cross contamination if disposable consumables are reused
- With a slightly damp cloth, clean the CO detector with a non-alcohol bactericide fungicide solution
- Do not breathe into the pipette (risk of contamination)

#### Disposal:

Used or dirty accessories must be disposed of with biological waste (DASRI).

#### Features:

Pipette	 Single use
Trapper	 Single use
Packaging	Compartmentalized sachets non sterile
Expiry date	See cardboard box label
Storage / transport	T°: 0°C - 50°C Humidity: 75% max.
Medical class	Class Im

First marketed: 2007

#### Labels:


 CE Marking Directive 93/42/CEE + N° of notified body

 Manufacturer identification

 Consult the accompanying documents

 Lot number

 Single use

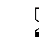 Expiry date